



TECHNICAL FEATURES

- Product size (L x W x H): 216 x 61 x 19cm
- Packing size: 130 x 61 x 26cm
- Packing size: 135 x 64 x 28cm
- N.W.: 20.5kg G.W.: 22.5kg
- Load bearing: less than 159kg

HOW TO USE

This Basket Stretcher can split it into two parts, making it compact and easy to transport. Its reliable fittings enable first-aid personnel to operate rapidly and safely. It has an adjustable feet-securing mechanism, safety belts and a mattress. All the materials used have dimensions suitable to work and they are fire-proof. They don't release any toxic or polluting substances and they are protected to obtain high resistance to wear and corrosion.

APPLIED FOR

With its special sling equipment, the stretcher is ideal for lifting and transport by helicopter. It is also designed for to be secured inside the helicopter.

CAUTIONS:

1. When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
2. In general operation, be careful not to tear out the surface of the stretcher.

MAINTENANCE

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

STORAGE & TRANSPORTATION

1. Store this product in the place of damp proof & non-corrosion.
2. Common transporting vehicle can carry this product.